

# OYSTA

SenseCare™



## Monitor:

- Temperature and Humidity (%)
- VIP Activity Pattern
- VIP Mobility and Activity

- Support Button
- Register carer visits
- Long Battery Life

## Additional Sensors:

- Flood/Leak Detector
- Bed/Chair Pad
- Door Sensor

## Introducing SenseCare

SenseCare is part of our SafeHome™ range. Our SafeHome range are passive sensors that aid the Care Circle in delivering preventative care.

SenseCare is quick and simple to install. It's a hub-less sensor which monitors Vulnerable Independent People's (VIPs) homes and detects any anomalies in the home environment and VIP activity patterns.

SenseCare connects to IntelliCare™ and when the sensor detects anomalies, it notifies the care circle in IntelliCare. Welfare checks can be conducted to prevent any possible anomalies from expanding to complex issues.

Through the data collected from SenseCare, IntelliCare can provide insight into a VIP's wellbeing which can help the care circle identify the need for preventative care

The **Care Circle** can use the SafeHome range to ensure a VIP's home environment stays within healthy parameters for them and to show that VIPs are mobile and active.

Trends can be detected with SenseCare and through IntelliCare, the care circle can be notified and take action to prevent the condition worsening.

## Benefits for the Care Circle

### Temperature and Humidity

VIPs are more prone to illness when there are lower temperatures and high humidity levels.

SenseCare can have safe temperature and humidity levels, pre-set. SenseCare sends the information to IntelliCare™ daily and if it breaches unsafe level limits, the care circle is notified so they can act and conduct a welfare check.

### Movement

SenseCare includes Passive Infrared (PIR) sensors that detect movement within a room so you can monitor a VIP's activity pattern and ensure they are mobile. Lack of movement could indicate a decline in a VIP's health which you'll be able to identify using the analysis within IntelliCare™.

### Support Required Button

When the blue button on SenseCare is pressed, a silent 'Support Required' event is sent to the chosen contact or Alarm Receiving Centre (ARC).

The care circle can use the button to register their visits by pressing it, twice.

### Battery

SenseCare's battery lasts a minimum of two years, however, the life of SenseCare can be extended should it be plugged in using the supplied USB-C power cable.



Measure Temperature and humidity



Monitor VIPs activity pattern



Register a care visit



Battery lasts a minimum of two years

## Take SenseCare further

More intelligence enhances the ability for the care circle to provide preventative care. SenseCare can expand its intelligence reach further with additional accessories.

### Bed Pad

The bed pad will detect the VIPs sleep pattern. Should there be unusual activity, such as frequent disturbances in the night, this may indicate underlying health issues, so IntelliCare will send a notification to the care circle.

### Chair Pad

Should your VIP be spending too long sitting in one place, there could be an underlying health reason and any early prevention such as simply encouraging movement will benefit them. IntelliCare will notify you of any unusual activity.

### Door Sensor

The door sensor can tell when a door is left open, which is useful if VIPs experience forgetfulness, so the care circle can contact them, to prompt them to close the door.

Additionally, in conjunction with SenseCare's movement PIR, it can tell whether a person has entered or exited the room, which can highlight to the care circle that the VIP is active in their home.

### Leak Detector

The leak detector will let the care circle know (through IntelliCare) if the property is at risk of a leak or flood so you can act.

## About SafeHome™

Oysta helps Vulnerable Independent People (VIPs) stay safe at home and away from home.

SenseCare is a passive sensor which is part of our SafeHome™ range, designed to keep VIPs safe when they are at home and aid the care circles in delivering preventative care.

With the SafeHome range, you can monitor VIPs:

- Temperature
- Humidity
- Activity
- Sleep pattern
- Mobility pattern
- Water intake

You'll be able to:

- Remotely tune their smoke alarm
- Respond when support is required
- Register care visits
- React to fire hazards

Our sensors link into our easy-to-use care platform, IntelliCare™. IntelliCare is used by the care circle to identify trends and analysis to support effective care decisions.

Oysta's IntelliCare platform simplifies technology enabled care and transforms reactive care to personalised, preventative care, helping customers to deliver better outcomes.



# OYSTA

OYSTA TECHNOLOGY

Home Ground Barn, Pury Hill Business Park, Towcester, NN12 7LS, UK.

Tel +44 (0) 1295 530 101 Email [info@oysta-technology.com](mailto:info@oysta-technology.com) [www.oysta-technology.com](http://www.oysta-technology.com)

Version: Aug2022

